



5000W STRONG POWER

4 HANDLES WORK SIMULTANEOUSLY

BUILD MUSCLE
AND SCULPT YOUR BODY
No SWEAT REQUIRED!





HIFEM Technology



BEFORE

Skin, fat and muscle from your overall body appearance.

DURING

The skin remains unaffected while the energy penetrates to fat and muscle levers.

AFTER



This results in substantial improvement of your appearance due to building muscle and burning fat.





Using High Intensity Focused Electro-Magnetic (HIFEM) technology, Emsculpting Pro induces supramaximal muscle contractions not achievable through exercise (or voluntary contractions).

During normal voluntary muscle contractions the muscle fibers relax between each nervous stimulus. Emsculpting Pro generates impulses at such a rapid frequency that it doesn't allow such a relaxation phase. Supramaximal contractions cause an enhanced adrenaline release, which triggers an intensive lipolysis response in fat cells. When exposed to the supramaximal contractions, the muscle tissue is forced to adapt, responding with a deep remodeling of its inner structure that results in muscle building and fat burning.


Mechanism of muscle contraction

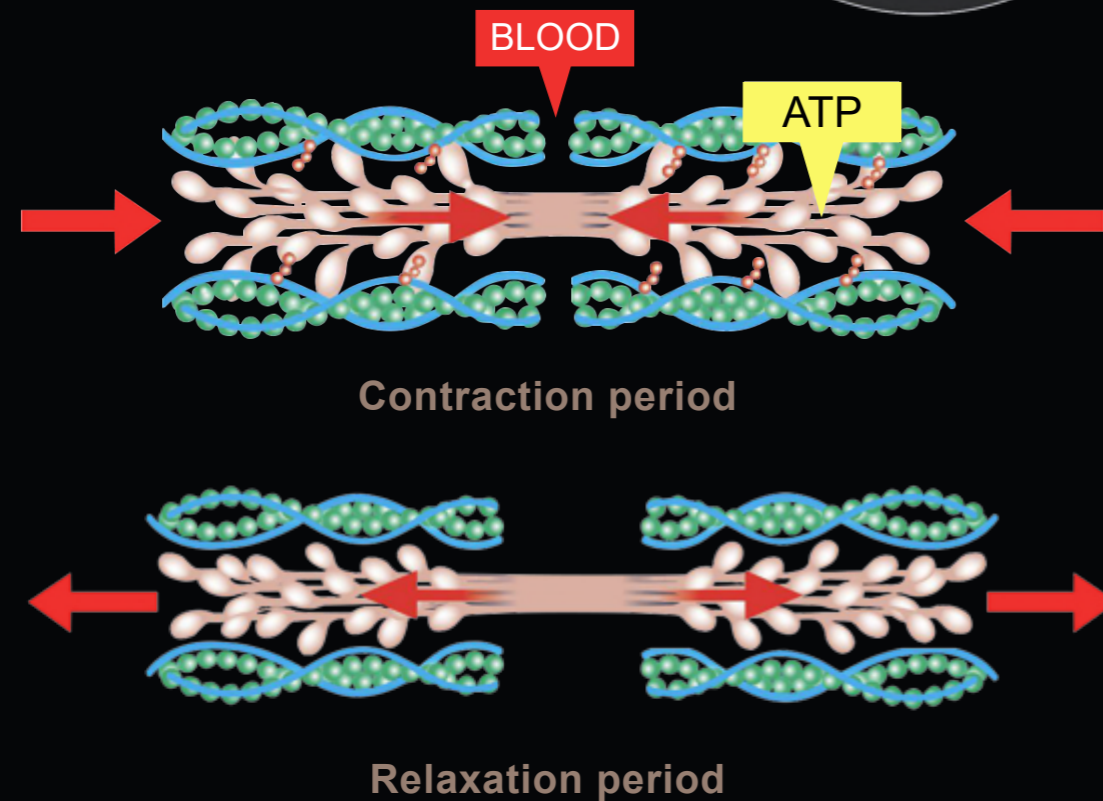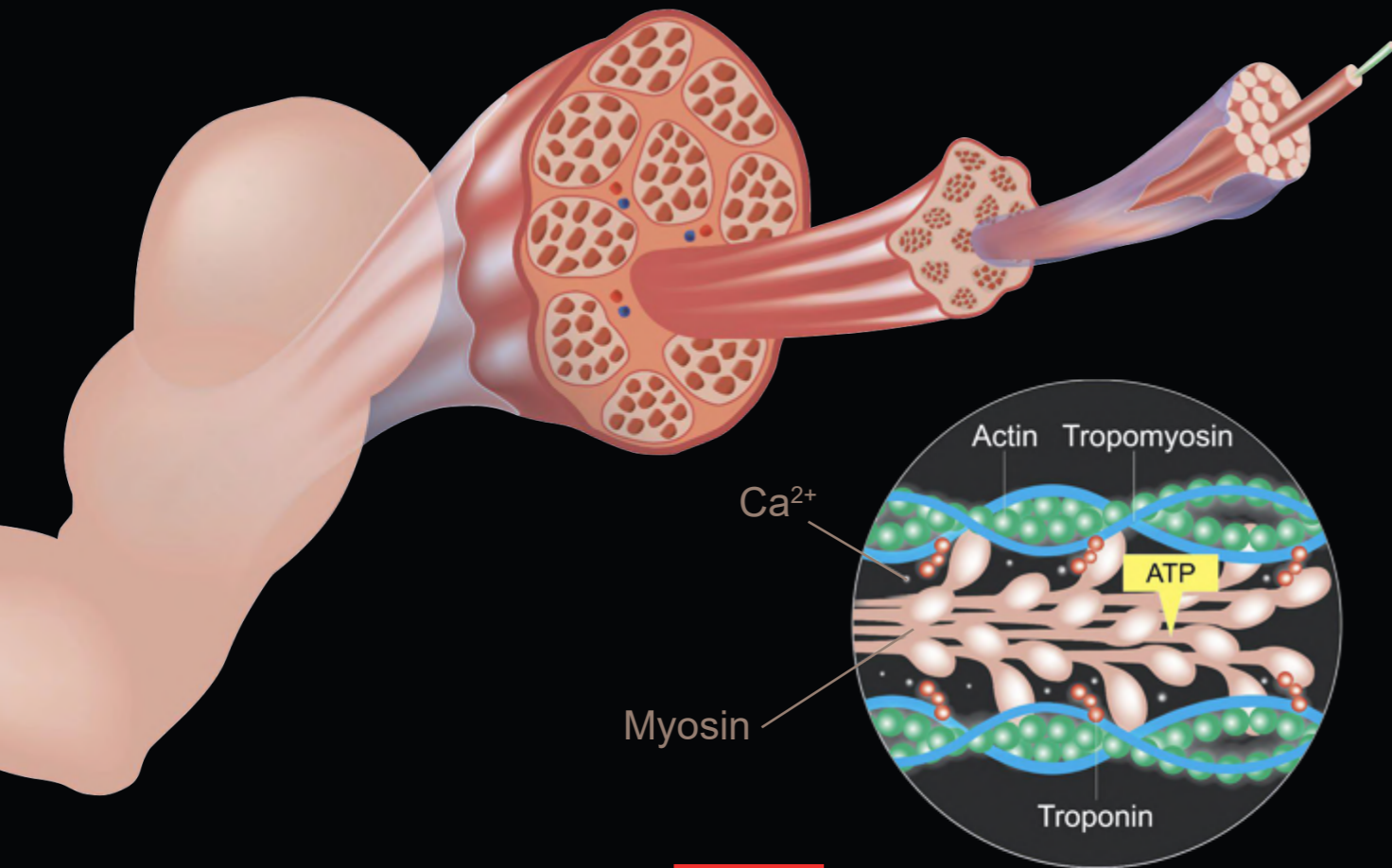

Nerve impulses set off a biochemical reaction that causes myosin to stick to actin

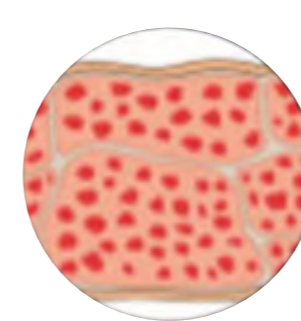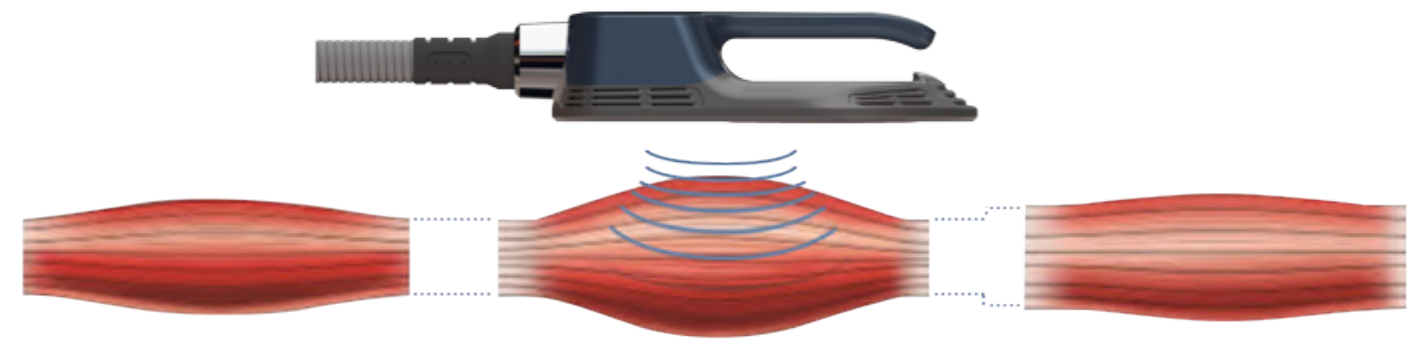


Lax muscle with decreased muscle density and volume.

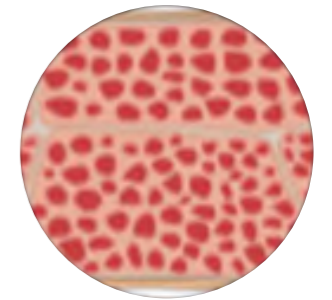

The process results in increased muscle density and volume.

Emsculpting Pro induces very strong (supramaximal) muscle contractions, not achievable through workout. When exposed to these strong contractions, the muscle tissue is forced to adapt to such extreme condition. The muscle starts remodelling its inner structure by multiplication of fibers and their growth. The HIFEM uses a specific range of frequencies that does not allow muscle relaxation between two consecutive stimulations. The muscle is forced to remain in contracted state for multiple seconds. When repeatedly exposed to these high load conditions the muscle tissue is stressed and is forced to adapt. Recent studies reported that on average 15%- 16% increase in abdominal muscle thickness was observed in treated patients one to two months after HIFEM treatments.

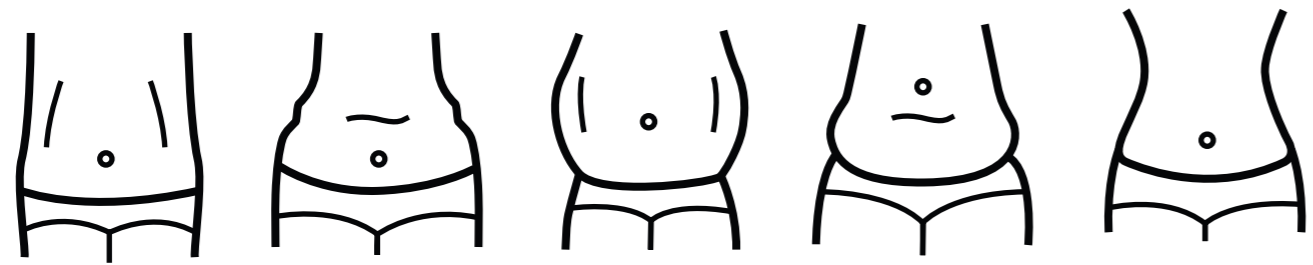


H Type

Folding Type

Fat Type

After Childbirth

Build abdominal muscles Perfect Firm abs and V line abs

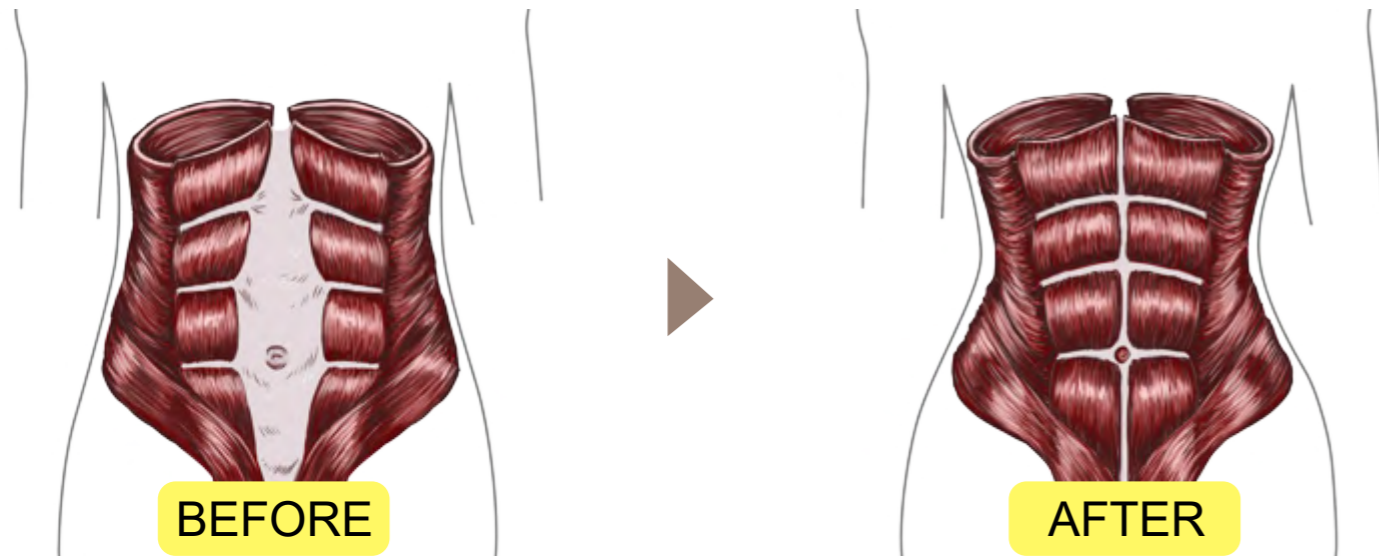



The Emsculpting Pro works on the abdomen area contracting and tightening the muscles to both burn fat and sculpt the area. This treatment can help you improve the muscle definition that you are already seeing from your fitness routine. It is also ideal for addressing that stubborn inch of fat on your stomach that just won't go away.



Emsculpting Pro

for the abdomen



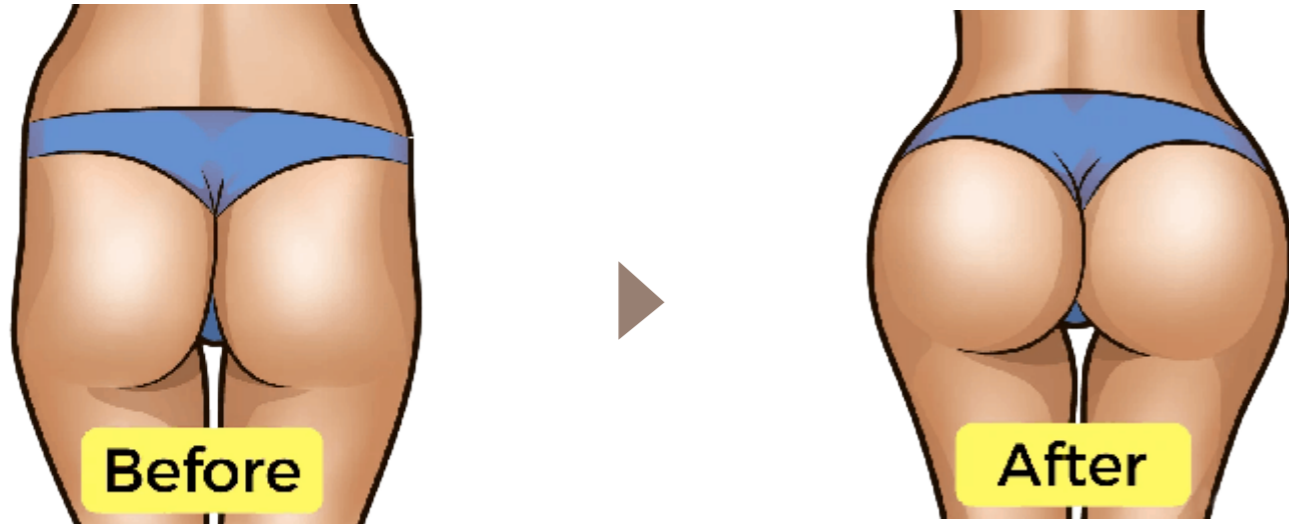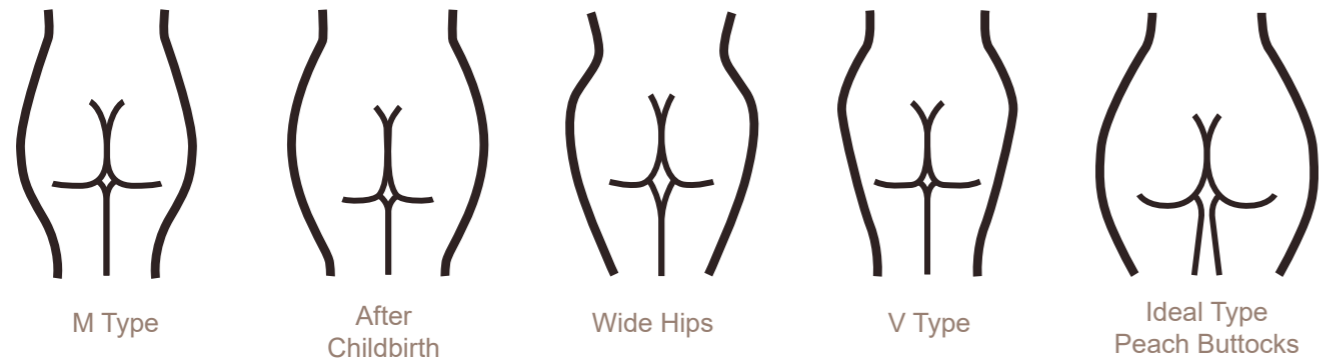

Emsculpting Pro works on the buttocks as well, using the same supramaximal muscle contractions as with the abdomen area. Many patients opt to try this treatment in place of an invasive Brazilian butt lift. HIFEM technology delivers the world's first noninvasive butt lift procedure. This treatment can give you the curvature, tone, and lift that you desire. It is an alternative to the Brazilian Booty Lift procedure and provides a defined shape instead of adding excess fat to the area.



Emsculpting Pro


for the buttock



New Curved Applicator Cintours Arms, Thighs & Calves



The Curved handle is sure to be welcomed by the thousands of men and women who want a little help with contouring the arms with stronger, more defined triceps and biceps and individuals who want to tone, lift, and sculpt the upper regions of their legs.

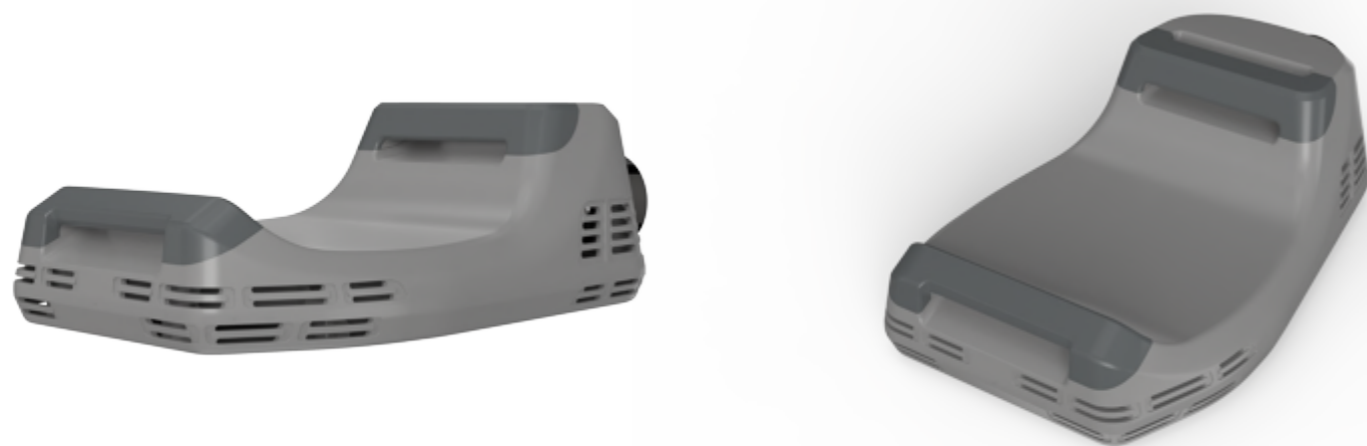

Emsculpting^{Pro}





16% ↑
Increase In Muscle Mass

19% ↓
Reduction Of Fat

4 cm (1.5 inches) loss in waistline circumference

11% reduction in abdominal muscle separation (called diastasis recti)

5 x increase in fat metabolism (apoptotic index increased from 19% to 92% after treatment)

80% patients with visible lifting effect of buttocks

96% patient satisfaction



FAQ

Q A candidate?

A Anybody

Q A downtime?

A Not at all

Q How long will it last?

A Extremely long

Q How often?

A 2~3 times a week

Q Additional supplies?

A No consummables

Q How long can I use it?

A No limit

Q How long?

A 30 minutes per session

Q How many time?

A 6~8 treatment recommended

Q Manual program?

A Easy & Effective manual mode



Clinic



Gym

ANYWHERE Shape Your Body ANYTIME

Emsculpting pro has a wide range of application scenarios, which can be used in beauty clinics, gyms and even offices or homes to shape your perfect body anytime, anywhere



Home



Office

[CONTACT US](#)

SPECIFICATIONS

Frequency:	1-150hz
Power:	5000w
Intensity:	7Tesla
Screen:	12.4inch tough screen
Pulse width:	310 μ s \pm 10%
Cooling:	Air Cooling
Voltage:	AC220V \pm 10% 10A 50HZ , 110v \pm 10% 10A 60HZ